Manufacturer: Safe Life Corporation Model Tested: A430 (LOT # 091110) Date Tested: May 13, 2020 Report Prepared: May 14, 2020

These findings pertain to the Safe Life Corporation, A430 (LOT # 091110) submitted for testing, and may not be applicable to other stockpile facilities and/or under different environmental storage conditions. The maximum and minimum filter efficiency was 95.53% and 84.80%, respectively. One respirator measured more than 95% efficiency. Nineteen respirators measured less than 95% efficiency. There was no designated expiration date for these models by the previously held NIOSH approval holder.

NIOSH regulation sets the minimum quality and performance requirements for the approval of respirators (42 CFR 84). **NIOSH does not have requirements for shelf life or storage conditions for particulate-only APRs.** The approval holder (i.e. the entity that is granted the approval from NIOSH) is responsible for understanding how their products' design or performance may be affected by various use or storage conditions and must provide instruction for establishing the proper use, storage, and maintenance procedures for their approved products, which may include designating a shelf life. FFR or particulate filter packaging (such as the box) often includes NIOSH-approved user instructions, label information, and recommendations on shelf life. Additionally, some approval holders also disseminate recommendations related to storage and shelf life through resources such as user and web notices. The respirators tested in this study were generally not designed for long-term storage.

Based on research conducted by NIOSH and this limited testing, NIOSH does not have enough information to definitively know the level of protection that may be provided by respirators that 1) are stored for prolonged periods of times; 2) are stored under various storage conditions; or 3) have exceeded the approval holder's designated shelf life. Users of respirators that have exceeded the designated shelf life should be forewarned to avoid a false sense of confidence; these devices may not provide the same level of protection as those that have not exceeded the designated shelf life. We recommend contacting the approval holder(s) of the respirators in the stockpile with specific questions regarding the use of product beyond the manufacturer- designated shelf life.

The results provided in this letter are specific to the subset of NIOSH-approved N95s, past their designated shelf life, that were provided to NPPTL for evaluation.

These results will be added to the CDC guidance for <u>Stockpiled N95 Filtering Facepiece Respirators</u> <u>Beyond the Manufacturer-Designated Shelf Life</u>.

Evaluation of Stockpiled and Beyond Manufacturer-Designated Shelf Life N95s



Test: TEB-APR-STP-0059


Manufacturer: Safe Life Corporation

Item Tested: A430 (LOT # 091110)

Expiration Date: None Provided

Manufacture Date: None Provided

Filter	Flow Rate (LPM)	Initial Filter Resistance (mmH ₂ O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency (%)
1	85	9.7	5.64	6.40	93.60
2	85	11.8	4.14	4.47	95.53
3	85	9.6	11.2	12.3	87.70
4	85	12.1	12.7	12.9	87.10
5	85	10.7	10.8	11.0	89.00
6	85	10.0	12.8	15.2	84.80
7	85	11.8	8.15	8.15	91.85
8	85	9.3	10.4	12.2	87.80
9	85	14.0	8.04	8.04	91.96
10	85	11.4	7.01	7.12	92.88
11	85	12.2	5.84	5.99	94.01
12	85	8.4	14.1	14.4	85.60
13	85	9.2	7.65	7.79	92.21
14	85	11.2	6.54	6.83	93.17
15	85	8.4	12.3	14.6	85.40
16	85	8.3	12.5	15.1	84.90
17	85	10.6	4.85	5.35	94.65
18	85	9.4	11.3	13.3	86.70
19	85	10.8	4.83	5.57	94.43
20	85	10.6	4.99	5.58	94.42
N	linimum Filter Effici	iency: 84.80	Maxi	mum Filter Efficien	cy: 95.53

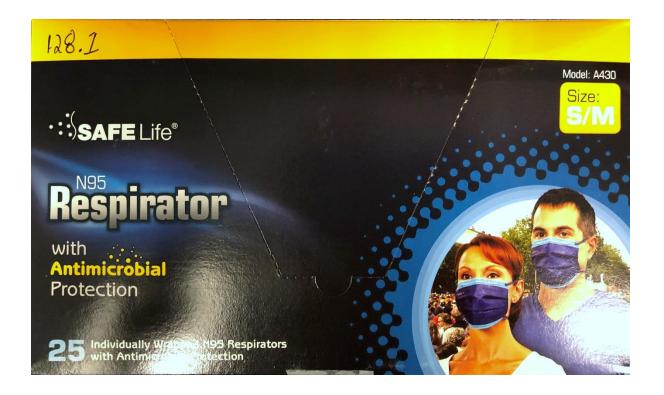

				1.
	N95			N95
	499	1 Ha	tor	2. A -
		Prin C		B - C -
W	ith <mark>Anti</mark> m	icrobial F	rotection	J - I M - acc N -
	12250 El Cam	Life Corp. ino Real, Suite 350	Notional Institute for Occupational Safety and Health	in t 0 -
		o, CA 92130 365-5084	NOSH	and P -
1	866-1	500-0004	A DESCRIPTION OF THE OWNER OWNER OF THE OWNER OWN	
and the second s	866.1	503-3004		S - Ref
These Res			n The Following Configuration:	Ref
These Res	pirators Are A			Ref :]
These Res	pirators Are A	pproved Only I		Ref
7 A.	pirators Are A RESI	pproved Only II PIRATOR COMP	PONENTS	Ref Ma Sal

Individually Wrapped N95 Respirators with Antimicrobial Protection

ction

ulate Filter (95% filter efficiency level) effective against particulate of oil; time use restrictions may apply.

ons and Limitations

- se in atmospheres containing less than 19.5 percent oxygen.
- use in atmospheres containing less than 19.5 percent oxygen. use in atmospheres immediately dangerous to life or health. xoeed maximum use concentrations established by regulatory standards. properly use and maintain this product could result in injury or death. oved respirators shall be selected, fitted, used, and maintained in with MSHA, OSHA, and other applicable regulations. bisbitute, modify, add, or omit parts. Use only exact replacement parts uration as specified by the manufacturer. User's Instructions, and/or maintenance manuals for information on use ance of these respirators.

ance of these respirators. pes not evaluate respirators for use as surgical masks. r Critical User's Instructions and/or specific use limitations apply. 's Instructions before donning.

by nino Real, Suite 350 A 92130 prp.com

Made in USA DB A430 Rev1.1

a registered trademark of Safe Life Corp. © Safe Life Corp.



