Manufacturer: Dongguan Sure-on Industrial Ltd., (Makrite Industries, Inc.) Model Tested: 9500-N95; Lot# 060320 Date Tested: October 6, 2020

These findings pertain to the Dongguan Sure-on Industrial Ltd., (Makrite Industries, Inc.), model 9500-N95; Lot# 060320. The packaging and labeling indicate that it is a NIOSH-approved product, under approval number TC-84A-5411. The packaging also indicates that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator).

Ten respirators were submitted for evaluation. The samples were tested using a modified version of NIOSH Standard Test Procedure (STP) TEB-APR-STP-0059. This modified assessment plan can be found <u>here</u>.

The maximum and minimum filter efficiency was 98.95% and 91.84%, respectively. Nine respirators measured more than 95%. One respirator measured less than 95%.

This product has head bands/straps. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirators represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for <u>Crisis Capacity Strategies (during known</u> <u>shortages)</u>.

Evaluation of International Respirators

Test: Modified TEB-APR-STP-0059

Date Tested: October 6, 2020

Report Prepared: October 6, 2020

Manufacturer: Dongguan Sure-on Industrial Ltd., (Makrite Industries, Inc.)

Item Tested: 9500-N95; Lot#060320

Country of Certification: China (GB2626), USA (claimed, 42 CFR 84)

Filter	Flow Rate (LPM)	Initial Filter Resistance (mmH ₂ O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency (%)	
1	85	13.3	1.79	1.79	98.21	
2	85	14.4	4.17	4.17	95.83	
3	85	14.1	2.70	2.70	97.30	
4	85	18.1	1.09	1.09	98.91	
5	85	18.7	1.50	1.50	98.50	
6	85	11.5	8.16	8.16	91.84	
7	85	15.0	1.05	1.05	98.95	
8	85	17.1	1.63	1.63	98.37	
9	85	17.1	2.20	2.20	97.80	
10	85	15.4	1.08	1.08	98.92	
Minimum Filter Efficiency: 91.84%			Maximum Filter Efficiency: 98.95%			

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.



Pictures have been added to the end of this report.



ife Holds Unlimited Possibilities

N95 Particulate Respirators

LIMITED WARRANTY IMPORTANT NOTICE TO PURCHASER:

This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, expressed or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Makrite will be met by products sold.

Exclusive Remedies: damages for the breach of this limited warranty are limited to the replacement of such quantity of Makrite, products proved to be defectively manufactured. Except as provided above, Makrite shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to use products by the user.

WARNINGS:

li

1. Wearer must read fitting instructions and use limitations before use.

9500-N95

NIOSH Approved Product

Particulate

Respirators

1.1.1

- 2. This respirator is not guaranteed to protect against the risk of contracting a disease or infection.
- 3. Before occupational use of this respirator, a written, respiratory protection program must be implemented meeting all the local government requirements. In the United States employers must comply with OSHA 29 CFR 1910,134 which includes medical evaluation, training, and fit testing.

Made in China

RESPIRATOR FITTING INSTRUCTION



STEP 1

Hold the respirator in hand with the nosepiece at your fingertips, allowing the headbands to hang freely below your hand.



STEP 2

Press the respirator firmly against your face with the nosepiece on the bridge of your nose.



STEP 3

Stretch and position the top band high on the back of the head. Stretch the bottom band over the head and position below your ears.



STEP 4 Using both hands, mold nosepiece to the shape of your nose.



STEP 5

To test fit: a) Cup both hands over the respirator being careful not to distrub position, and b) inhale vigorously. If air leaks around the edges, reposition the straps or adjust strap tension for better fit.

Please carefully follow these fitting instructions during each use to achieve proper fit.

Made in China





HIMAN SERVIC	MAH 11F- Xinta New	KRITE INDUSTR 5, No.79, Sec. 1 ai 5th Rd., XiZhi Taipei City 221,	, Dist., , Taiwan	A settional institute for Decomparison of Statesy	SH°
CEPARTNERS	7 Tel:	+886-2-2698-24	19		
This respirato		in the following conf			
TC-	Protection ¹	Respirator 9500-N95	Cautions and Limita	ations ²	
84A-5411	N95	X	ABCJMNOP		
	1. Prot	tection	÷		
			level)effective		
		may apply			
2. Cautions and					
A - Not for use	in atmospheres o	containing less than	19.5% oxygen.		
A - Not for use B - Not for use C - Do Not exc	in atmospheres of in atmospheres in atmospheres ir eed maximum us	mmediately danger e concentrations es	ous to life or health.	ry standards	
A - Not for use B - Not for use C - Do Not exc J - Failure to pr M - All approve	in atmospheres of in atmospheres in eed maximum us operly use and m of respirators shal	mmediately danger e concentrations es naintain this product Il be selected, fitted	ous to life or health. stablished by regulato t could result in injury , used, and maintaine	or death.	
A - Not for use B - Not for use C - Do Not exc J - Failure to pr M - All approve with MSHA N - Never subs	in atmospheres of in atmospheres in eed maximum us operly use and m d respirators shai , OSHA and othe titute, modify, ado	mmediately danger e concentrations es laintain this product Il be selected, fitted r applicable regulati d, or omit parts. Use	ous to life or health. stablished by regulato t could result in injury , used, and maintaine	or death. ed in accordance	
A - Not for use B - Not for use C - Do Not exc J - Failure to pr M - All approve with MSHA N - Never subs configuration	in atmospheres of in atmospheres in eed maximum us operly use and m d respirators shal , OSHA and othe titute, modify, ado n as specified by	mmediately danger e concentrations es naintain this product II be selected, fitted I or applicable regulati I, or omit parts. Use the manufacturer.	ous to life or health. stablished by regulato t could result in injury , used, and maintaine ions e only exact replacem	or death. ed in accordance ent parts in the	
A - Not for use B - Not for use C - Do Not exci J - Failure to pr M - All approve with MSHA N - Never subs configuration O - Refer to use maintenance	in atmospheres of in atmospheres in eed maximum us operly use and m id respirators shall , OSHA and othe titute, modify, ado n as specified by ers'instructions, a of these respirator	mmediately danger e concentrations es laintain this product II be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. nd/or maintenance s.	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa	or death. ed in accordance ent parts in the	
A - Not for use B - Not for use C - Do Not exci J - Failure to pr M - All approve with MSHA N - Never subs configuration O - Refer to use maintenance	in atmospheres of in atmospheres in eed maximum us operly use and m id respirators shall , OSHA and othe titute, modify, ado n as specified by ers'instructions, a of these respirator	mmediately danger e concentrations es naintain this product Il be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. nd/or maintenance	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa	or death. ed in accordance ent parts in the	
A - Not for use B - Not for use C - Do Not exci J - Failure to pr M - All approve with MSHA N - Never subs configuration O - Refer to use maintenance	in atmospheres of in atmospheres in eed maximum us operly use and m id respirators shall , OSHA and othe titute, modify, ado n as specified by ers'instructions, a of these respirator	mmediately danger e concentrations es laintain this product II be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. nd/or maintenance s.	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa	or death. ed in accordance ent parts in the	
A - Not for use B - Not for use C - Do Not exc J - Failure to pr M - All approve with MSHA N - Never subs configuration O - Refer to use maintenance P - NIOSH does	in atmospheres of in atmospheres in eed maximum us operly use and m d respirators shal , OSHA and othe titute, modify, ado n as specified by ers'instructions, a of these respirator not evaluate respirator	mmediately danger e concentrations es iaintain this product ll be selected, fitted r applicable regulati f, or omit parts. Use the manufacturer. ind/or maintenance s. ators for use as surgio	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa cal masks.	or death. ed in accordance ent parts in the	
A - Not for use B - Not for use C - Do Not excl J - Failure to pr M - All approve with MSHA N - Never subs configuration O - Refer to use maintenance P - NIOSH does	in atmospheres c in atmospheres i eed maximum us operly use and m d respirators shal , OSHA and othe titute, modify, add n as specified by ers instructions, a of these respirator not evaluate respirator	mmediately danger e concentrations es laintain this product ll be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. nd/or maintenance s. ators for use as surgio	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa cal masks.	or death. ed in accordance ent parts in the tion on use and	
A- Not for use B - Not for use C - Do Not excl J - Failure to pr M - All approve with MSHA N - Never subs configuration O - Refer to use maintenance P - NIOSH does SERVICE 1. Do not use th	in atmospheres c in atmospheres in eed maximum us operly use and m d respirators shal , OSHA and other titute, modify, add n as specified by ers'instructions, a of these respirator not evaluate respirator TIME LI	mmediately danger e concentrations es laintain this product ll be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. Ind/or maintenance s. ators for use as surgion MITATION or for more than 8	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa cal masks.	or death. ed in accordance ent parts in the tion on use and	
 A- Not for use B - Not for use C - Do Not exclude J - Failure to pr M - All approve with MSHA N - Never subsconfiguration O - Refer to use maintenance P - NIOSH does SERVICE Do not use the in dirty workpl Leave contamproper fit camproper fit camp	in atmospheres c in atmospheres c operly use and m d respirators shal , OSHA and other titute, modify, add n as specified by ers'instructions, a of these respirator not evaluate respirator ETIME LI e same respirator laces that could ninated area and not be obtained	mmediately danger e concentrations es laintain this product ll be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. Ind/or maintenance s. ators for use as surgi MITATION or for more than 8 result in high filter d remove respirator or the respirator is	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa cal masks.	or death. ed in accordance ent parts in the tion on use and or intermittent, nes difficult or or distorted.	
 A- Not for use B - Not for use C - Do Not exclude J - Failure to pr M - All approve with MSHA N - Never subsconfiguration O - Refer to use maintenance P - NIOSH does SERVICE Do not use the dirty workpl Leave contain proper fit can po not use the 3. The maximum	in atmospheres c in atmospheres c in atmospheres in eed maximum us operly use and m d respirators shall , OSHA and other titute, modify, add n as specified by ers instructions, a of these respirator not evaluate respirator active state respirator laces that could ninated area and not be obtained the same respirator not be obtained on the same respirator	mmediately danger e concentrations es laintain this product II be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. Ind/or maintenance s. ators for use as surgi MITATION or for more than 8 result in high filter d remove respirator or the respirator is or again once it h	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa cal masks. hours, continuous of r loading. or if breathing becom s damaged, soiled, of as been used in cor five (5) consecutive	or death. ed in accordance ent parts in the tion on use and or intermittent, hes difficult or or distorted. taminated area	
 A- Not for use B - Not for use C - Do Not exclude J - Failure to pr M - All approve with MSHA N - Never subsconfiguration O - Refer to use maintenance P - NIOSH does SERVICE Do not use the dirty workpl Leave contain proper fit can po not use the 3. The maximum	in atmospheres c in atmospheres c in atmospheres in eed maximum us operly use and m d respirators shall , OSHA and other titute, modify, add n as specified by ers instructions, a of these respirator not evaluate respirator active state respirator laces that could ninated area and not be obtained the same respirator not be obtained on the same respirator	mmediately danger e concentrations es laintain this product ll be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. Ind/or maintenance s. ators for use as surgi MITATION or for more than 8 result in high filter d remove respirator sor again once it h this respirator is 1	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa cal masks. hours, continuous of r loading. or if breathing becom s damaged, soiled, of as been used in cor five (5) consecutive	or death. ed in accordance ent parts in the tion on use and or intermittent, hes difficult or or distorted. taminated area	
 A- Not for use B - Not for use C - Do Not exclude J - Failure to pr M - All approve with MSHA N - Never subsconfiguration O - Refer to use maintenance P - NIOSH does SERVICE Do not use the dirty workpl Leave contain proper fit can po not use the 3. The maximum	in atmospheres c in atmospheres c in atmospheres in eed maximum us operly use and m d respirators shall , OSHA and other titute, modify, add n as specified by ers instructions, a of these respirator not evaluate respirator active state respirator laces that could ninated area and not be obtained the same respirator not be obtained on the same respirator	mmediately danger e concentrations es laintain this product ll be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. Ind/or maintenance s. ators for use as surgi MITATION or for more than 8 result in high filter d remove respirator sor again once it h this respirator is 1	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa cal masks. hours, continuous of r loading. or if breathing becom s damaged, soiled, of as been used in cor five (5) consecutive	or death. ed in accordance ent parts in the tion on use and or intermittent, hes difficult or or distorted. taminated area	
 A- Not for use B - Not for use C - Do Not exclude J - Failure to pr M - All approve with MSHA N - Never subsconfiguration O - Refer to use maintenance P - NIOSH does SERVICE Do not use the dirty workpl Leave contain proper fit can po not use the 3. The maximum	in atmospheres c in atmospheres c in atmospheres in eed maximum us operly use and m d respirators shall , OSHA and other titute, modify, add n as specified by ers instructions, a of these respirator not evaluate respirator active state respirator laces that could ninated area and not be obtained the same respirator not be obtained on the same respirator	mmediately danger e concentrations es laintain this product ll be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. Ind/or maintenance s. ators for use as surgi MITATION or for more than 8 result in high filter d remove respirator sor again once it h this respirator is 1	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa cal masks. hours, continuous of r loading. or if breathing becom s damaged, soiled, of as been used in cor five (5) consecutive	or death. ed in accordance ent parts in the tion on use and or intermittent, hes difficult or or distorted. taminated area	
 A- Not for use B - Not for use C - Do Not exclude J - Failure to pr M - All approve with MSHA N - Never subsconfiguration O - Refer to use maintenance P - NIOSH does SERVICE Do not use the in dirty workpl Leave contamproper fit campon not use the indirty day 	in atmospheres c in atmospheres c in atmospheres in eed maximum us operly use and m d respirators shall , OSHA and other titute, modify, add n as specified by ers instructions, a of these respirator not evaluate respirator active state respirator laces that could ninated area and not be obtained the same respirator not be obtained on the same respirator	mmediately danger e concentrations es naintain this product ll be selected, fitted r applicable regulati d, or omit parts. Use the manufacturer. Ind/or maintenance s. ators for use as surgi MITATION or for more than 8 result in high filter d remove respirator sor again once it h this respirator is f eginning from the	ous to life or health. tablished by regulato t could result in injury , used, and maintaine ions e only exact replacem manuals for informa cal masks. hours, continuous of r loading. or if breathing becom s damaged, soiled, of as been used in cor five (5) consecutive	or death. ed in accordance ent parts in the tion on use and or intermittent, hes difficult or or distorted. taminated area	

Makrite Qualification 过滤口罩 Disposable Respirator 品名称: oduct Type 品型号: 9500-N95 Product Model 产品尺寸: 144mm*177mm*53mm(±10 Product Size 白色 56%涤纶短纤/18%不织布/16%像根带/ 56%PE/18% Polyester/16%Koitted Elastic Strap 060320 主要成份: Major Ingredient Strap / 8%Aluminium/2 se Foam 生产日期: 060320 **Production Date** 生产批号: Lot number 有效期:三年 Period Of Validity 3 years 检验员: Q003 Inspector 生产企业: 东莞迅安塑胶纤维制品有限公司 Manufacturer Dongguan Sure-on Industrial Ltd 生产地址: 广东省东莞市塘厦镇诸佛岭工业区 Address No.18, Guangyuan St., 168 Industrial Area, Zho Fo Ling, Tang Xia, Dongguan City, China 联系电话: 0769-87723170 执行标准: 美国NIOSH N95/GB 2626-2006 Executive Standard 本工厂通过ISO 9001:2015国际质量管理体系认证 The factory has passed ISO 9001:2015 international quality management system







